





radiography. It should be part of the basic equipment of any radiographic school. The phantom contains a real human skeleton as well as outlines of larynx, lung, heart and kidneys (organs will create a shadow on the image), which allows taking real X-ray images like in a patient. Using a real skeleton provides even smallest guiding structures which is impossible with a plastic skeleton. During assembly of this phantom we pay special attention to the correct size of joint spaces. All joints are moveably mounted allow positioning in all normal x-ray positions (e.g. frog position, pro- and supination of lower arm). The arms can be moved upwards which makes the phantom suitable for use in all kinds of osseous examinations under CT. Each phantom is hand-made one of a kind; it may differ in size and appearance. Depending on the individual phantom it may have some pathologies, outer shape may differ depending on size of the skeleton. The new version was re-designed in co-operation with a well-known German school for radiographers and fits all needs for education in radiography. This phantom is only sold against proof of medical use. Life size.

Also Partial phantoms available!

Ref.no. 7200 Details:



Radiographic positioning doll, plastic skeleton

This model offers all features of model 7200 but includes a plastic skeleton and is due to this only suitable for positioning training. Including a transport and storage case.

Ref.no. 7201 (ohne Abb.) Details: 画狀系圖

Got a broken phantom? Have it repaired at our premises in 4 easy steps:

1. Pick-Up Service
The forwarder comes
with an empty transport
case and collects
your phantom.



2. Repair Cost Estimate
We will provide you
with a cost estimate.
You decide about a
repair then.



3. Repair
The phanotm will be repaired fast and reasonable priced at our factory.
Result: as NEW!



4. Delivery
The "new" phantom
will be delivered by
the forwarder.



X-Ray thorax

Thorax X-ray phantom, consisting of spine (C6 to L3), thoracic cage, shoulder blades and collar bones, embedded in soft materials. Mounted without metal parts. Organs (lung, heart, kidneys) are present as outlines to create a shadow on the x-ray image. The natural bones used may have, depending on availability, individual pathological changes, minor structural defects or mounting holes.

Ref.no. 7400





X-Ray Phantom Pelvis, flexible

This flexible X-Ray phantom Pelvis contains real human bones and offers the unique opportunity to take real xray images. This Phantom consist of a complete pelvis including coccyx, two lumbar vertebrae (L4+L5) and femur stumps. The soft material makes this product ideal for palpation. The natural bone used may have, depending on availability, individual pathological changes, minor structural defects or moun-Ref.no. 7410

ting holes. Delivered with strong transport case.



X-ray phantom head

Human skull, safely embedded in plastic for easy use. The jaws are slightly open to allow dental panoramic images of the teeth. The neck includes some cervical vertebrae depending on the ordered type. An embedded tread allows the use with a tripod. The jaw may have dental gaps, dental repairs, broken teeth, replaced teeth or other individual features. For detailed information about the available phantoms please contact our sales team.

X-ray phantom head with cervical vertebrae, transparent

Ref.no. 7300

X-ray phantom head with cervical vertebrae, opaque

Ref.no. 7310

X-ray phantom head, transparent

Ref.no. 7320

Ref.no. 7330

opaque

head,



Tripod for X-Ray Phantom head

Very strong tripod for use with the x-ray phantom head. Foldable but stable, holding the head safely in place. With rotatable head for precise positioning of the phantom in the x-ray machine

Ref.no. 7350

X-RAY PART **PHANTOMS**





Hand with wrist.

Transparent Ref.no. 7210

Opaque Ref.no. 7215





X-ray Phantom knee

Part of upper- and lower leg and patella.

Transparent Ref.no. 7250

Opaque Ref.no. 7255



X-ray Phantom lower arm

Hand with forearm and elbow. **Transparent**

Ref.no. 7220

Opaque Ref.no. 7225





X-ray Phantom elbow

Part of upper and lower arm. **Transparent** Opaque

Ref.no. 7260

Ref.no. 7265

Transparent Ref.no. 7340

Opaque Ref.no. 7345

X-ray Phantom shoulder

Humerus with shoulder girdle.

X-RAY PHANTOMS



Transparent Ref.no. R16700









Dental Radiography Head Phantom

This radiography phantom has removable jaws and tongue allowing for a variety of applications for training and research.

Features:

- Each tooth is individually modeled and has a threelayer structure of enamel, dentin and pulp cavity.
- Each hard tissue (enamel, dentin, cortical bone) and cancellous bone) has a particular HU number and X-ray absorption rate.
- Jaws and tongue are detachable to allow access to the oral cavity, pharyngeal cavity and maxillary sinus. Sensors, simulated lesions, or residue can be set in these cavities.
- ✓ Carotid arteries are prepared as lumens to accommodate simulated calcifications.

Anatomy:

- Synthetic skull with nasal cavity, maxillary sinus, mandible alveolar, and maxillary alveolar; cervical vertebrae and hyoid bone, teeth with enamel, dentin and pulp cavity.
- Tongue, oral cavity, pharyngeal cavity and carotid arteries.





Head with closed mouth Ref.no. R16525





Thorax

Includes a thoracic skeleton with embedded heart and lungs to provide realistic imaging. The scapulae are rotated outside of the lung fields for proper PA chest imaging.

Transparent Ref.no. R16702

Details:

Opaque

Ref.no. R16703 Details:



Pelvis

Includes lumbar / sacral spine, pelvic bony anatomy and proximal femurs.

Opaque

Ref.no. R16704 Details:







Right Elbow

Movable. Normal flexion range allows for AP/lateral and partial flexion views with one phantom.

Transparent Ref.no. R16705

Details:

Opaque Ref.no. R16706 Details:

Details:







Right Knee

Freely movable patella and joint allows for realistic positioning of the knee for AAP/lateral, oblique, sunrise and tunnel views.

Transparent

Ref.no. R16711 Details:

Details:

Opaque

Ref.no. R16712

Details:











- abducting to a horizontal position.
- Left shoulder rotates forward, up to a horizontal position.
- Elbows bend inward to approx. 90 degrees.
- Hip joints rotate forward up to 90 degrees, then rotate outward up to 45 degrees, respectively.
- Knees bend to approx. 90 degrees.
- The phantom can be held in the supine frog leg position.
- The limbs and head are detachable at joints and neck for wider applications.
- The head supporter facilitates various head positions.

- Hands and feet with bone trabeculae
- Lungs with pulmonary vessels
- Mediastinal space
- / Liver
- Kidneys

Phantom materials:

- Radiology absorption and Hounsfield number approximate to human body.
- Phantom size: approx. 165 cm height Phantom weight: approx. 50 kg







Fractured hand with forearm for R16900

Hand with forearm showing bone fractures for radiography. Can be replaced against the original hand with forearm of R16900. Of course this phantom can also be used as stand-alone type too.

Ref.no. R16900-1



Pediatric whole body phantom

The new pediatric whole body phantom is molded after a 5-year-old child of 105 cm height. This phantom is a life-size, full body anthropomorphic phantom with a state-of-the-art synthetic skeleton, lungs, liver, mediastinum and kidneys embedded in soft tissue substitute.

Training skills:

- √ Plain X-ray photography and basic CT scanning.
- Basic patient positioning for X-ray and CT.

Features

- Movable joints allow basic positioning for plain X-ray.
- Training and research applications can be enriched by disassembling the phantom into 10 individual parts (head, limbs and trunk).
- / The phantom has no metal parts or liquid structures.
- Main joints have life-like articulation, allowing various positioning for training.
- / Life size synthetic skeleton
- / Hands and feet with bone trabeculae
- Lungs with pulmonary vessels
- Mediastinal space
- / Liver
- / Kidneys

Seperates into:

- / head
- √ trunk
- / right upper arm
- √ right forearm with hand
- / left upper arm
- / left forearm with hand
- / right thigh
- / right lower leg with foot
- / left thigh
- √ left lower leg with foot

Ref.no. R16970

Details:










Peadiatric full body phantom

Full body phantom as R16970, but additionally with fractures that are typical for child abuse. All fractures are prepared on the left side and show for example a spiral fracture of the distal tibia or a forearm shaft fracture.

Ref.no. R16975





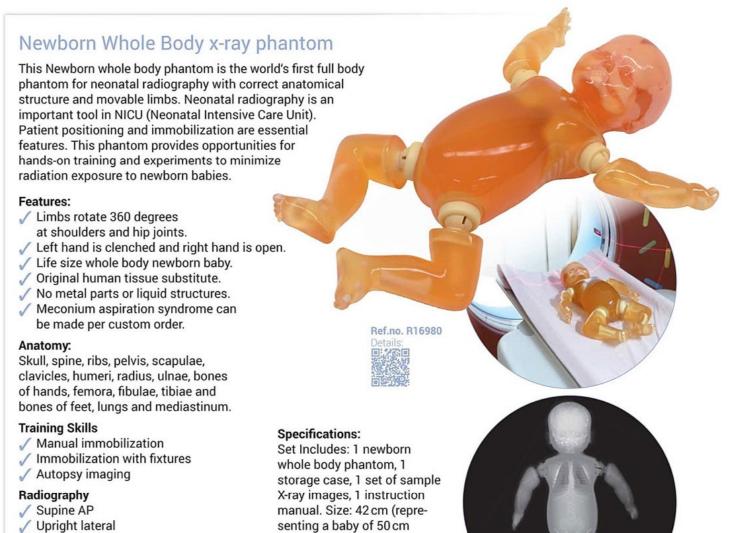

height) weight: 2.8 kg.

Supine lateral

Whole body CT phantom

A unique, life size whole body phantom for CT provides a variety of educational application as well as visual evaluation in finding out optimal scanning conditions. The phantom can also be used for plain X-ray, showing life-like images. No metal parts or liquid structure are used. Main joints have close-to human articulation, allowing various positioning for training. The phantom can be disassembled into 10 parts. Improved shoulder joint system enables the phantom to take arm-up position. Organs are anatomically correct and have appropriate HU numbers.

Patient positioning:

Shoulders: rotate through a full 360 degrees in the sagittal plane, approx. 180 degrees to side-ways.

Hip joints: rotate forward up to approx. 90 degrees, then abduct up to 45 degrees each.

- Knees: bend up to approx. 90 degrees.
- ✓ Elbows: bend up to approx. 90 degrees.
- The phantom can be held in the supine frog leg position.
- The limbs and head are detachable at joints and neck for wider applications.
- The head supporter facilitates various head positions.

Phantom materials:

Radiology absorption and Hounsfield number approximate to human body.

Internal Organs:

- Head and Trunk
- √ Synthetic skull
- ✓ Cervical vertebrae
- √ Brain
- √ Vertebrae
- ✓ Clavicles
- / Ribs
- √ Sternum
- √ Scapula
- Coxal bones
- √ Femurs
- Lungs with pulmonary vessels

- √ Trachea
- Liver with portal and hepatic veins
- / Pancreas
- / Kidneys
- Gallbladder
- √ Spleen
- / Aorta
- Vena Cava
- ✓ Ureter
- Urinary bladder
- / Prostate
- / Rectum
- Sigmoid Colon














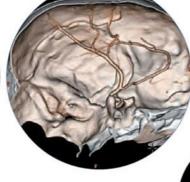





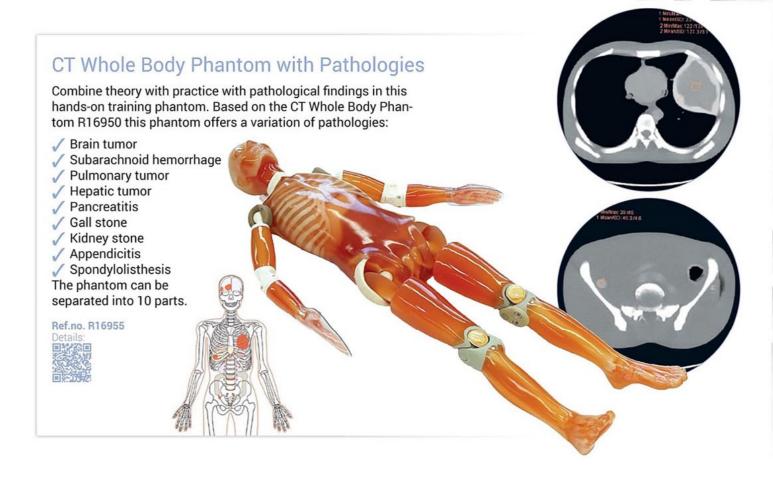


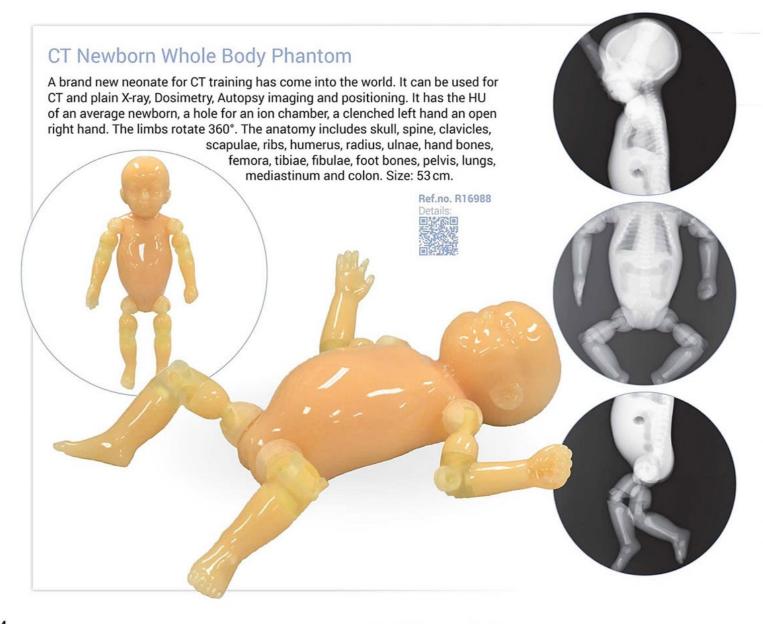



CT abdomen phantom

This unique anthropomorphic upper abdomen phantom allows obtaining CT images approximate to clinical data. The elaborate anatomy of organs allows a multi-dimensional approach. Liver, portal vein, bile duct, hepatic vein, hepatic artery, kidneys, pancreas spleen and IVC are embedded along with synthetic bones. Each individual organ has a particular Hounsfield number close to human organ. Embedded anatomical structures are Lungs (no internal structure), Heart (no internal structure), liver, portal vein, bile duct, hepatic vein, hepatic artery, kidneys, pancreas, spleen, Ref.no. R16513 IVC, spinal column, ribs. Vessels and organs with a contrast agent can be included as a special order.

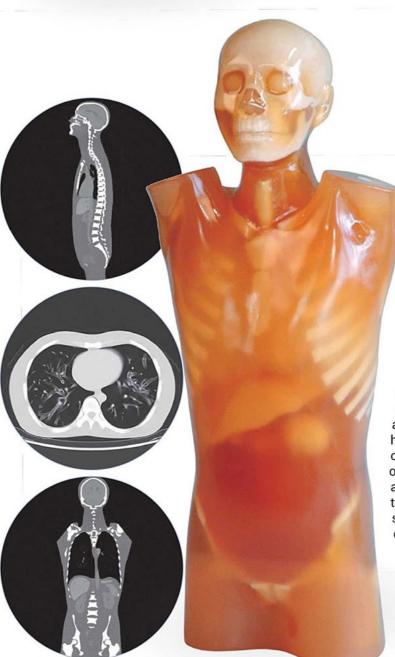

CT torso phantom

A one-piece anthropomorphic torso phantom with anatomical structures allows various CT approaches including helical scanning. Along with state-of-the-art synthetic bones, brain with cerebral ventricles, eye balls, lungs with three dimensional pulmonary vessels, trachea, liver with portal and hepatic veins, kidneys, gallbladder, pancreas, spleen, aorta, cava, ureter, urinary bladder, prostate, rectum, sigmoid colon are embedded. Each individual organ has particular Hounsfield number which corresponds that of human body. The original phantom material with radiation

absorption approximate to human tissue allows scanning under actual clinical setting.

Ref.no. R16512 Details:





Chest plates for R16511

For simulation of an over weighted patient.

Ref.no. R16510-1 Details:

Components for Radioisotope

The set of RI container inserts can be set in the chest phantom in place of the standard inserts allowing wider research applications including PET/CT fusion evaluation.

Lungs of urethane Ref.no. R16511-1

Details:

Liver RI container Ref.no. R16511-2 Details:





Gallbladder RI container Ref.no. R16511-3

Details:

Pulmonary nodule RI container

Ref.no. R16511-4 Details:



Mediastinum with left myocardium RI container

Ref.no. R16511-5



Subsolid Nodules Simulation Set

The Subsolid Nodules Simulation Set is a set of simulated nodules designed for study and training in Ground-Glass Opacity (GGO) detection and interpretation. Both mixed and pure GGO are provided in a variety of sizes and HU numbers. The set also includes 3D GGO modelled on clinical CT data. The simulated nodules can be attached to the pulmonary vessels on the chest phantom R16511.



3D GGO Ref.no. R16511-10 Details:















No. 1-7, Concentric Ref.no. R16511-6

Ref.no. R Details:

No. 11-12, Eccentric Ref.no. R16511-8

Details:

No. 8-10, Eccentric Ref.no. R16511-7 Details:

No. A - h, Pure GGO Ref.no. R16511-9 Details:

Pediatric chest phantom

Imaging and dosimetry for radiosensitive 5-year-old. Chest X-ray is one of the most common examinations in pediatric radiography. This Pediatric Chest Phantom is designed to find out optimal parameter and protocols to minimize radiation exposure to children. The phantom has two kinds of interchangeable lung inserts. The lung vascular insert can be used to study image quality in relation to CT / X-ray protocols. The lung density insert allows users to evaluate dosage distribution in the lung field.

Features:

- Two types of interchangeable lung inserts are available. lung vascular insert and lung density insert.
- Pencil-shaped ion chamber for CTDI can be set in the mediastinum.
- TLD or RPL dosimeters can be set in the thyroid block and the Lung density insert.
- Lung vascular inserts with pulmonary vessels provide life-like radiographs.
- Detachable internal structure allows insertion of variety of pathologies and targets.
- / Simulates a life-size chest of 5- year-old.



Anatomy:

Rib, clavicle, spine, mediastinum, scapula, sternum and pulmonary vessel (lung vascular insert only)

Ref.no. R16515

Applications:

- √ Pediatric Chest X-ray
- / Pediatric Chest CT
- Dosimetry

Das Set beinhaltet:

- 1 five-year-old chest torso
- √ main body: synthetic bones are embedded
- / thyroid block
- / diaphragm block
- 1 lung density insert: mediastinum, lung fields (L+R)
- 1 set of sample images
- / 1 instruction manual
- √ Size: 32 x 17 x 38 cm, weight: 6 kg

Lung cancer screening model


This Phantom is a CT phantom developed to facilitate optimizing the radiation dose and other scanning conditions for lung cancer screening CT examination with Helical CT or MDCT, which is aiming at early detection of lung cancers. As the screening is usually done on healthy people, the necessity of minimizing the exposure while maximizing the image quality is considered to be particularly high. The phantom is designed to set conditions for detection of small early lung cancers such as GGA, which are difficult to be found by plain X-ray. Anthropologic structure of the phantom provides life-like images allowing operators visual evaluation, while quantitative evaluation on radiation dose and density curve of the image can be done stimulatory with a single scanning.

The model consists of a life size torso with arm up position and has the following internal structures:

- ✓ Bones
- Simulated tumors on sections of three lung area:
- / Apical portion of the lungs
- Bifurcation of the trachea
- ✓ Base of lungs
- Dose meter hole (13 mm dia., on the central axis of the phantom)
- 1 8-step linearity phantom
- 8 steps of 30mm dia. density samples are embedded

Ref.no. R16532 Details:

Details

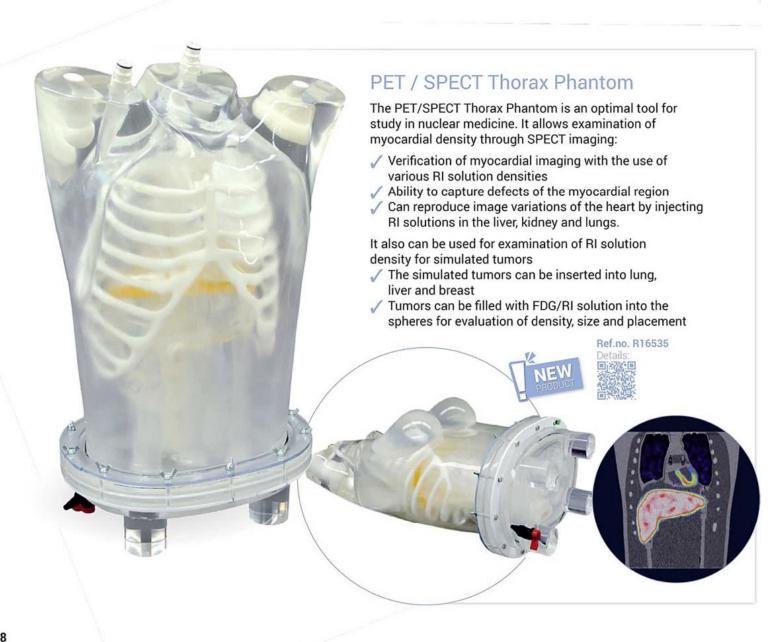

Radiation therapy phantom

This phantom is developed for the treatment planning and machine adjustment in the radiation therapy. The body consists of 3 cm slices with a 3 x 3 cm hole matrix for inserting glass dosimeters. The model material has a natural radioparency allowing the correct adjustment of the machines.

This makes it ideal for planning and machine adjustment. The phantom has a








This child phantom has different lifelike densities for tissue, bones and lungs. It is cut into slices and has holes for dosimeters. The size is 60cm. Delivery with holding and fixation frame.



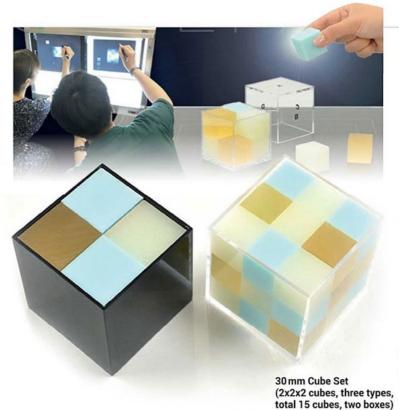


XCUBEFAN Radiology Cube Phantom

Designed for beginners to provide better understanding of the special characteristics seen in radiology imaging. The XCUBEFAN is a compact yet practical tool to educate learners on usage of x-ray equipment and its impact on the interpretation of diagnostics images. Learners can experience a large range of practice by stacking and repositioning different blocks with varied radiodensities. The black box is designed for practice, whilst the clear box facilitates visual explanation for instructors. There are three types of cubes, orange with CT value appr. 0 and density 1,06, yellow with CT value appr. 1000 and density 1,21 and blue with CT value appr. 500 and density 1,4.

Ref.no. R16945

20 mm Cube Set (3x3x3 cubes, three types, total 30 cubes, two boxes)

Ref.no. R16946

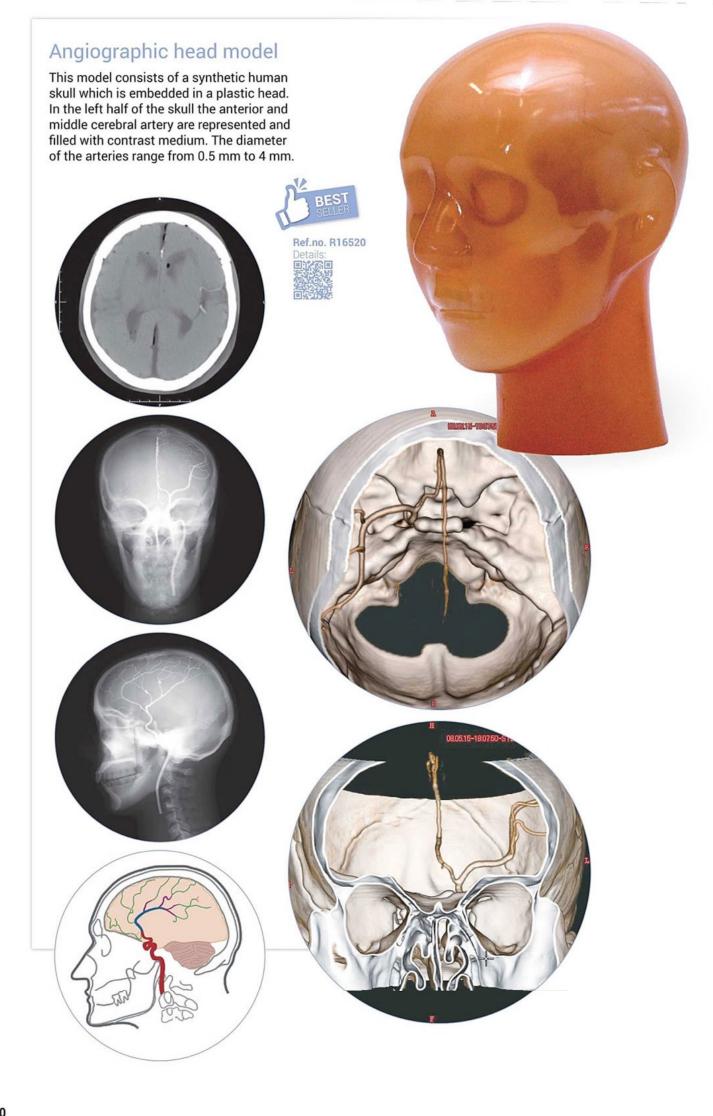

L4, L5, pelvis and femurs (partial). **Specifications:** Set Includes: 1 prostate phantom Size: 35 cm.

CT Knee Ligament Phantom Detailed Knee anatomy with HU of each bone, cartilage and ligament. This anthropomorphic knee phantom allows the visualization of ligaments and cartilage. Close-to-human radiation absorption and HU for each anatomical structure as well as realistic artifacts. It shows femur, tibia, fibula, articular cartilage of patella, meniscus, cruciate ligament, medial collateral ligament, fibular collateral ligament and articular cartilage. Can be used for CT and plain X-ray.

NATURE LINE CT PHANTOMS


Are created from real human patient data and are manufactured with latest technology. Bone and soft tissues are accurately represented with realistic Hounsfield units for all tissues at 120 kVp tube voltage CT imaging. If the phantom is primarily required for CT imaging at different tube voltages (e.g., 100 kVp), the calibration of Hounsfield units can be adapted accordingly upon request. The phantom also yields realistic tissue contrast in X-ray imaging. Air voids are filled with a cellulose-polymer composite of approximately -80 Hounsfield units

Custom nade orders possible



Head CTA-Phantom

The phantom provides a highly realistic simulation of a head and neck CT angiography (with arterial contrast medium enhancement) of a patient with an arteriovenous malformation on the right side. The phantom comes at the original patient size and includes the cervical spine until the baseplate of the fifth cervical vertebra.

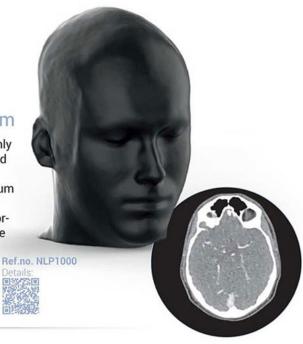

Hand-Phantom

The hand phantom provides an extremely realistic simulation of a hand. Pathologies (e.g. fractures, calcifications, bones and soft tissue tumors) can be integrated on request.



Abdomen-Phantom

The phantom provides a highly realistic simulation of a contrast medium enhanced abdomen (late arterial phase) of a patient. The phantom covers the abdomen from the tenth thoracic to the fourth lumbar vertebra. The liver and both kidneys are fully included.





Child Abdomen Phantom with a Vertebral Fracture

The phantom provides a highly realistic simulation of a contrast medium enhanced abdomen (portal venous phase) of a child with a vertebra plana (first lumbar vertebra). The phantom covers the abdomen from the tenth thoracic to the baseplate of the fourth lumbar vertebra. The liver and both kidneys are fully included.





Pelvis Phantom with Femoral Neck Fracture

The phantom provides a realistic simulation of a male pelvis without contrast medium enhancement and with a slightly dislocated femoral neck fracture on the left side. The phantom includes the entire pelvis and the fifth lumbar vertebra.

Ref.no. NLP1200 Details:



Pelvis Phantom with Coxarthrosis

The phantom provides a realistic simulation of a female pelvis without contrast medium enhancement and with advanced hip osteoarthritis on the right side. The phantom includes the entire pelvis and the fifth lumbar vertebra.


Ref.no. NLP1210 Details:



Foot-Phantom

The foot phantom provides an extremely realistic simulation of a foot. Pathologies (e.g. fractures, calcifications, bones and soft tissue tumors) can be integrated on request.

Ref.no. NLP1310 Details:

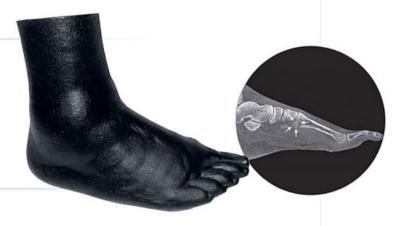



Breast Phantom for Mammography and Breast Tomosynthesis


Breast phantom with adipose and glandular tissue. This breast phantom was designed to simulate breast imaging in mammography and breast tomosynthesis. It represents a compressed breast of 4 cm thickness that can be placed under the compression


paddle. This phantom provides a realistic simulation of breast imaging. It was developed for testing and optimizing dose and image quality and for training of medical and technical staff. The phantom is manufactured from virtual data containing adipose and glandular tissue. The models are handmade unique piece, which can differ slightly in size and design. The phantom can be provided as one-piece anthropomorphic phantom or in a sectional design. Dosimeter openings and pathologic features can be included upon request.



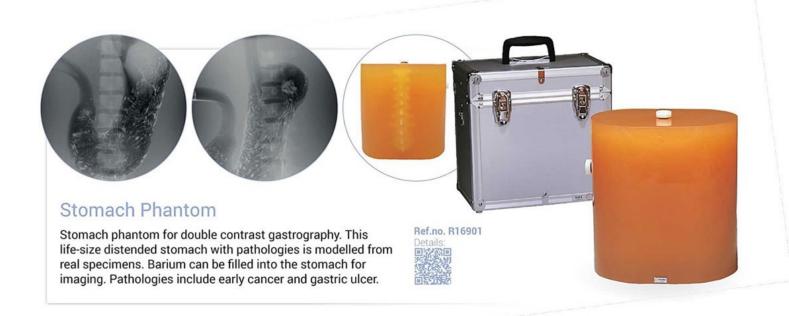

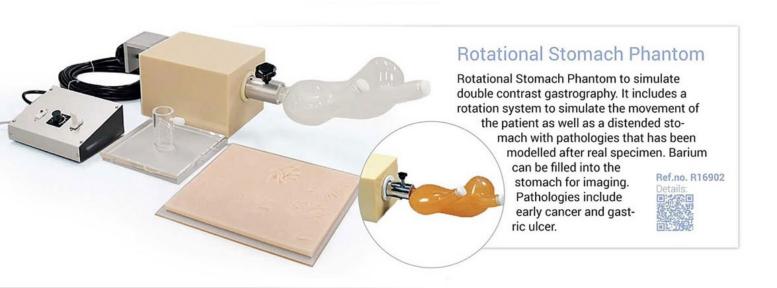

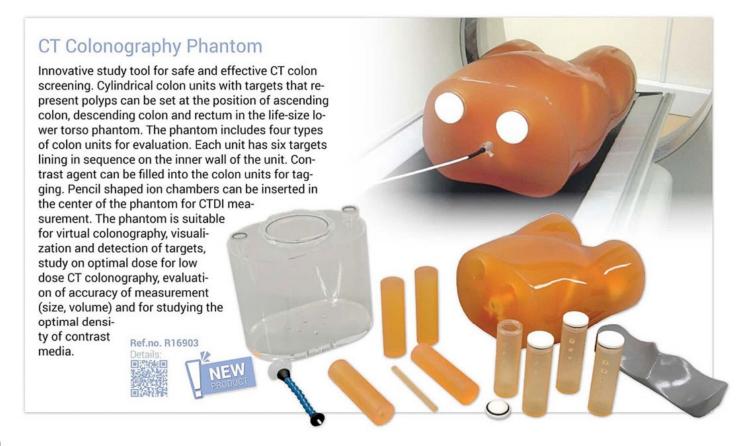

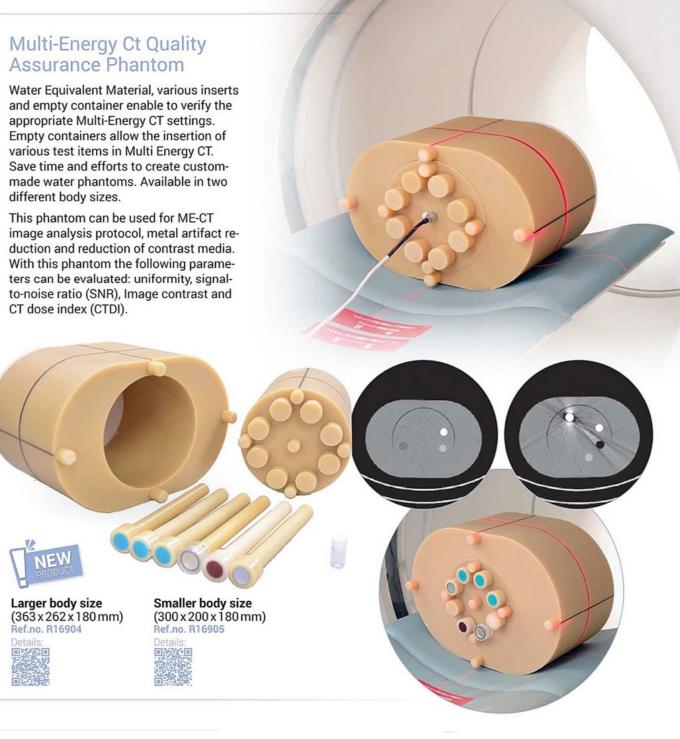




Bone Scintigraphy Quality Assurance Phantom


An innovative QA phantom for Bone Scintigraphy, Bone SPECT/CT and NaF-PET. The phantom can represent either thoracic or lumbar region by changing the filling of side cavities. Visual evaluation includes tumor detectability, image distortion and artifact. Quantitative

Evaluation such as Contrast and count ratio between vertebral body and tumor concentration linearity and recovery coefficient in the tumor, statistical noise, FWHM at the spinous process (relative index of resolution). Additionally verifica-

tion of scattering correction and attenuation correction is possible.

Ref.no. R16906 Details:

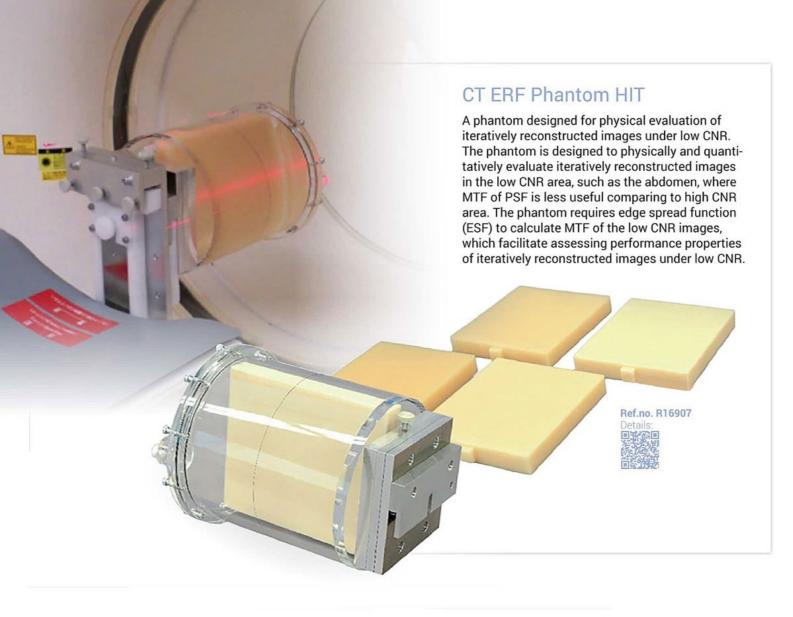

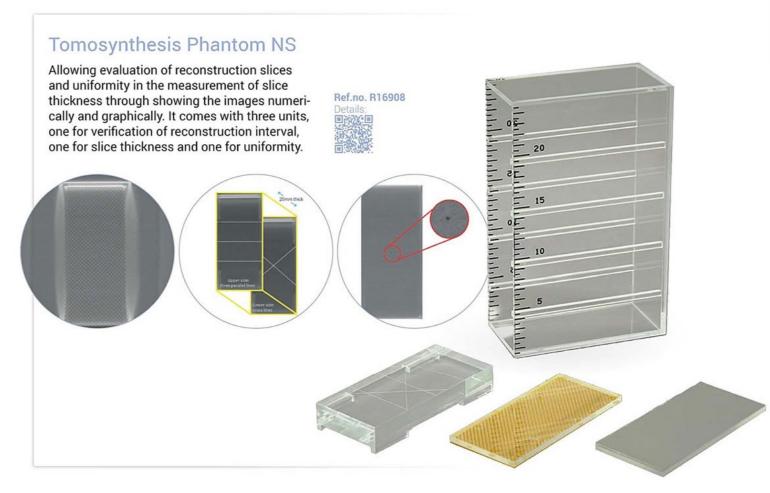



low contrast resolution, noise, mean.

Multi Slice CT Phantom MHT

This phantom can be used for features of CT evaluation such as high and low contrast resolution, feed direction and CTDI. The non-aqueous/easy setup allows a liquid-free evaluation session. This phantom is designed to allow evaluation in volume scanning. The evaluation parameters are: CTDI, contrast resolution, sensitivity profile, CNR, effective slice thickness and SSPz.















Evaluation of spatial resolution if simulated contrast enhanced vessels in CT. In each of the 5mm thick plates of this phantom there are 5 slits of 5mm length to represent vessels. Nine variations of vessel width are included and can be used in the phantom. The phantom can be used for evaluation of spatial resolution of simulated contrast enhanced vessels in CT.





CT-AEC Phantoms

Four types of phantoms designed for evaluation of CT-AEC performance.

- The image quality can be evaluated by noise and standard deviation on the phantom section images.
- Cone Phantom: evaluates performance of AEC for different patient sizes and gradual size changes in size along the axis
- Elliptical Cone Phantom: in combination with the Cone phantom facilitates evaluation of XY AEC
- √ Variable-XY Phantom: evaluates performance of XY
 AEC as cross section changes from circular to elliptical
- Stepped Phantom: evaluates the performance of the AEC to sudden changes in patient's cross section

The four phantoms can also be ordered individually:



Stepped Phantom Ref.no. R16915-4

Ref.no. R Details:



Variable-XY Phantom Ref.no. R16915-3

Details:



Elliptical Cone Phantom

Details:



Cone Phantom

Ref.no. R16915-1 Details:



Details:

Dynamic Cardiac CT Phantom MD-CT

For evaluation and research in ECG gating cardiac and thoracic CT. The heart model is made of human tissue substitute. Simulated coronary arteries including stenosis can be attached to the wall of the phantom heart. The phantom generates pulses that are synchronized with the cardiac movement for ECG gating. The controllable

parameters include the pulse rate, ejection volume and ejection fraction. The phantom is controlled by a touch panel.

Ref.no. R16916 Details:












This phantom represents movement of the heart, lungs and pulmonary nodule. The pulmonary nodule and the diaphragm move independently with the respiratory cycle. A three dimensional movement of the pulmonary nodule (linearly and rotationally) can be simulated. The respiratory movement of the abdomen is simulated as well as systolic and diastolic movement of the heart. Coronary arteries including stenotic examples are shown. The phantom can be connected to an ECG for ECG gating. The phantom can be used for respiratory gated

chest CT, tumor tracking in radiotherapy and ECG gated cardiac CT.

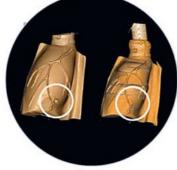





Ref.no. R16919

Digital Mammographic Phantom NCCE (fall)

This phantom is designed to evaluate CT-AEC performance. Image quality can be evaluated by noise and standard deviation on the phantom section images. The outer shape of the phantom simulates a compressed breast with D cup size. Targets include simulated microcalcifications, nylon fibrils, acrylic disks, an aluminium ring, Teflon disks, a Teflon ruler (slope) and a resolution test chart with 21 steps with different hydroxyapatite content. Steps range from 0 to 400 mg/cm, with 20mg/cm difference each.

BMD Chart Phantom UHA

Bone Mineral Density chart for microdensitometry (MD), 21 steps with different hydroxyapatite content. The steps range from 0 to 400 mg/cm, with 20 mg/cm difference each.



Tough Water Phantom

Human tissue substitute phantoms with water equivalent physical properties.



Ref.no. R16921

300 x 300 x 2 mm Ref.no. R16921A 300 x 300 x 3 mm Ref.no. R16921B 300 x 300 x 5 mm Ref.no. R16921C 300 x 300 x 10 mm Ref.no. R16921D 300 x 300 x 15 mm Ref.no. R16921E 300 x 300 x 20 mm Ref.no. R16921F 300 x 300 x 25 mm Ref.no. R16921G 300 x 300 x 30 mm Ref.no. R16921H 300 x 300 x 40 mm Ref.no. R169211 300 x 300 x 50 mm Ref.no. R16921J

400 x 400 x 2 mm Ref.no. R16921K 400 x 400 x 3 mm Ref.no. R16921L 400 x 400 x 5 mm Ref.no. R16921M 400 x 400 x 10 mm Ref.no. R16921N 400 x 400 x 15 mm Ref.no. R169210 400 x 400 x 20 mm Ref.no. R16921P 400 x 400 x 25 mm Ref.no. R16921Q 400 x 400 x 30 mm Ref.no. R16921R 400 x 400 x 40 mm Ref.no. R16921S 400 x 400 x 50 mm Ref.no. R16921T

Tough Bone Phantom

Human bone substitute phantoms to simulate body structure in combination with R16921 and R16923.



Ref.no. R16922

Compact bone

200 x 200 x 5 mm Ref.no. R16922A 200 x 200 x 10 mm Ref.no. R16922B 200 x 200 x 20 mm Ref.no. R16922C **Cortical bone**

200 x 200 x 5 mm Ref.no. R16922D 200 x 200 x 10 mm Ref.no. R16922E 200 x 200 x 20 mm Ref.no. R16922F 300 x 300 x 5 mm Ref.no. R16922J 300 x 300 x 10 mm Ref.no. R16922K 300 x 300 x 20 mm Ref.no. R16922L

Inner Bone


200 x 200 x 5 mm Ref.no. R16922G 200 x 200 x 10 mm Ref.no. R16922H 200 x 200 x 20 mm Ref.no. R16922I 300 x 300 x 5 mm Ref.no. R16922M 300 x 300 x 10 mm Ref.no. R16922N 300 x 300 x 20 mm Ref.no. R169220

Tough Lung Phantom

Human lung substitute phantoms to simulate body structure in combination with R16921 and R16922.



Ref.no. R16923

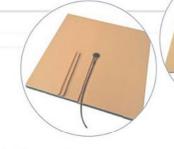

300 x 300 x 10mm Ref.no. R16923A 300 x 300 x 20mm Ref.no. R16923B 300 x 300 x 30mm Ref.no. R16923C 300 x 300 x 50mm Ref.no. R16923D



Water Body Phantom WAC

The water body phantom represents a human chest or abdomen to serve as radiation absorber and scatterer.

Ref.no. R16924

Acrylic Phantom XAC

Slab phantoms for radiation absorption and scattering measurement.

300 x 300 x 2 mm Ref.no. R16925B 300 x 300 x 3 mm Ref.no. R16925C 300 x 300 x 4 mm Ref.no. R16925D 300 x 300 x 5 mm Ref.no. R16925E

300 x 300 x 1 mm Ref.no. R16925A

300 x 300 x 8 mm Ref.no. R16925F

300 x 300 x 10 mm Ref.no. R16925G 300 x 300 x 20 mm Ref.no. R16925H


300 x 300 x 30 mm Ref.no. R16925I

300 x 300 x 40 mm Ref.no. R16925J

300 x 300 x 50 mm Ref.no. R16925K

300 x 300 x 80 mm Ref.no. R16925L

300 x 300 x 100 mmRef.no. R16925M

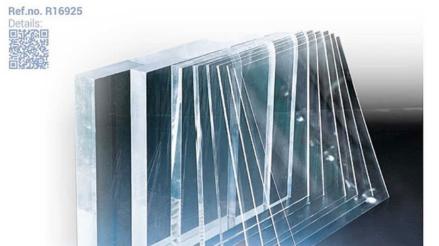

Contrast Detail Phantom

This phantom is used for image evaluation in plain x-ray. There are four types with different sizes and target types:





15 x 15 holes of depth range 1,0 bis 8,0 mm Ref.no. R16926A

10 x 10 holes of depth range 1,0 bis 5,5 mm Ref.no. R16926B

15x15 rods of height range 1,0 bis 8,0 mm Ref.no. R16926C

10x10 rods of height range 1,0 bis 5,5 mm Ref.no. R16926D

MRI Quality Assurance Phantom MHR/JMR II

This QA phantom for MRI allows to evaluate the slice thickness, spatial resolution, uniformity and geometric distortion as well as contrast. The Uniformity is maintained under the high magnetic field of 3.0 Tesla. The Uniformity provides high precision evaluation for other parameters. There are two types of this phantom.

The first phantom offers the following evaluation paramet

- √ Signal to Noise ratio (SNR)
- √ Image Uniformity
- ✓ RF uniformity
- √ Spatial resolution
- ✓ Spatial linearity (Image distortion)
- √ Slice thickness
- √ Slice position / separation
- √ mage contrast
- / Image artifact




Ref.no. R16927

Details:





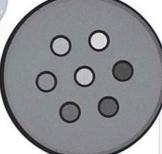



The second phantom offers the following evaluation parameters:


- √ Signal to Noise ratio (SNR)
- √ Image Uniformity
- √ Slice Thickness
- √ Spatial resolution
- √ Geometric distortion
- √ Ghost artifact
- √ Image contrast

JMR II Ref.no. R16928







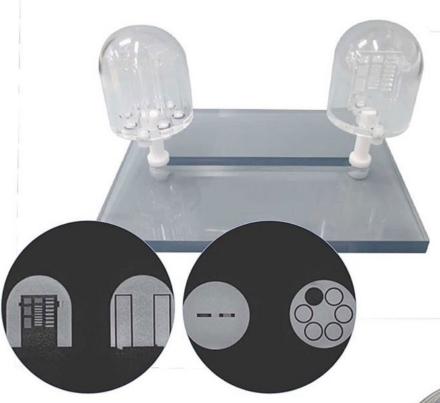

MRI Breast QA Phantom

An innovative phantom with the shape of breasts for detailed QA in Breast MRI. It allows quantitative evaluation of breast MRI with breast coils. It has an adjustable height of the phantoms in the range of 10cm to fit the depth of the coils. The horizontal position of the phantoms can be set aribitrarily on the 30cm long slit. The test parameters are spatial resolution and quantitative evaluation Ref.no. R16929


of ADC on test pieces of tissue substitute.

MRI Head Phantom NH

This life-size head phantom can be used to assess uniformity. It can be used for MRI, SPECT/ CT and CT. Comes with nickel chloride solution. Complies with JIS Z 4924.

Ref.no. R16930



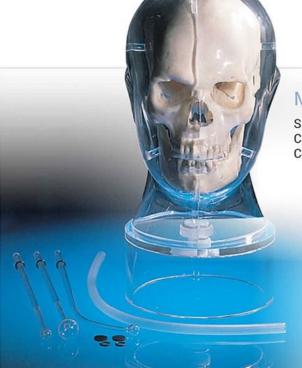



MRI / NM Head Phantom BHC


Simulates life-size head images in MRI and NM. Can be used in MRI, SPECT/CT and CT. Comes with Nickel Chloride Solution.





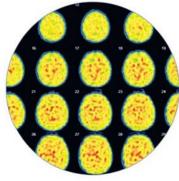

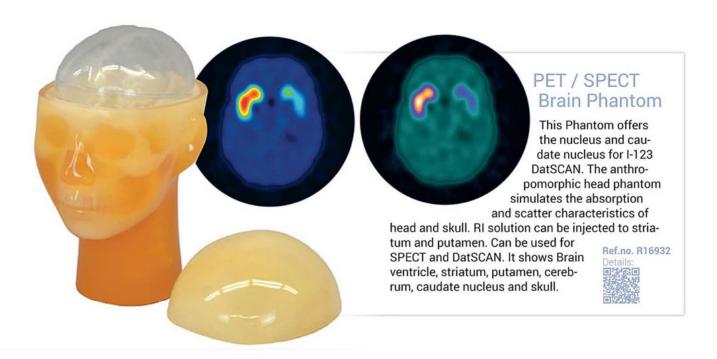


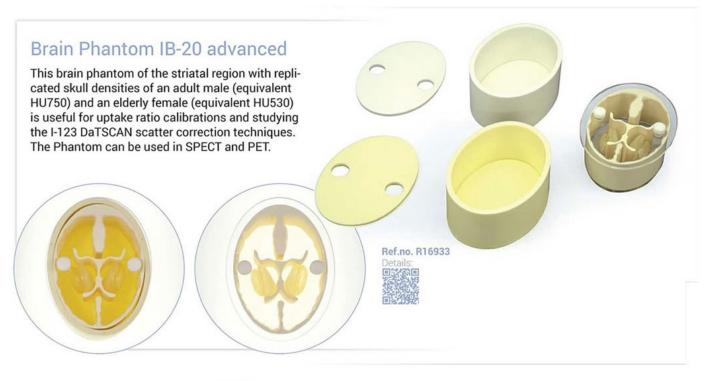




Brain Phantom Set IB-10

This set consists of 4 parts, a simulated skull unit, a brain unit comprising artificial grey and white matter, ventricular cavities and orbits. The set also incudes a Jaszczak phantom and a sectional phantom. Radioactive solutions may be added to the phantoms components. It can be used for the following evaluation parameters in SPECT and PET: Homogeneity evaluation, detectivity of grey matter and white matter, cross calibration, spatial resolution of negative images, Gamma ray absorption rate by skull and radioactive concentration and linearity of SPECT value.

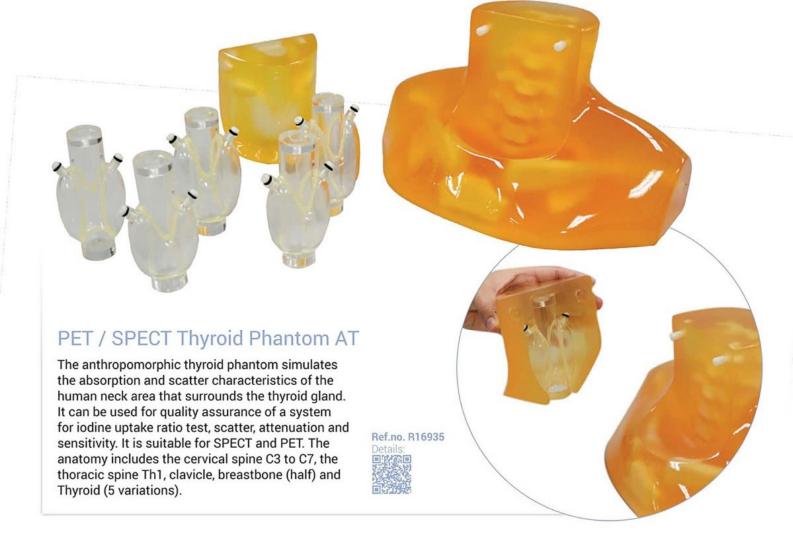

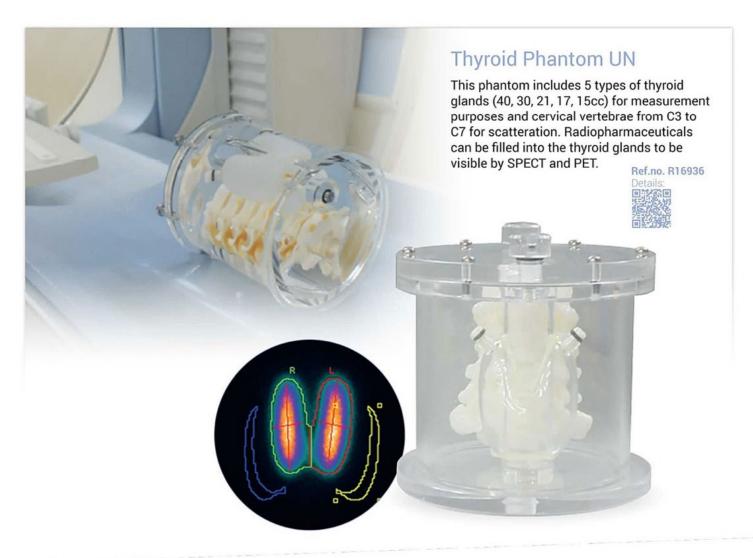


ORINS Thyroid Phantom IST

Oak Ridge Institute for Nuclear Studies Type phantom for measurement of thyroid radionuclide uptake. The neck phantom has cavities for iodine-131.

Ref.no. R16937



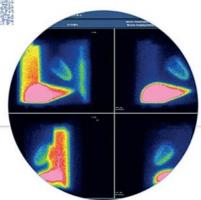





Myocardial Phantom HL

This phantom is designed for the study of high radio accumulation interference in the liver with the myocardial SPECT images. It allows the study of RI liver intake and its effect on the myocardial SPECT. A cold defect can be set in the left cardiac muscle. The background can be set individually in the lung field, mediastinum and right ventricle.

Ref.no. R16938

Details:

ECT HotCold Phantom SP-6

Volumetric measurement phantom for PET/SPECT. Five sphere containers with different sizes can be filled with RI solution. Volume of sphere phantoms are: 50 mm (100%), 80%, 60%, 40% and 20%. Can be used for SPECT, PET.

Ref.no. R16939





SPECT QA Phantom Set

A set of test units for daily QA of SPECT/PET. It consists of:

- Outer phantom
- Line source phantom
- ✓ Cold spot phantom
- ✓ Hot spot phantom
- Dose linearity phantomGeometric distortion phantom

The evaluation parameters are uniformity, dose linearity, spatial resolution and image distortion.

Ref.no. R16940








Makes the phantom suitable for your specific scanner.










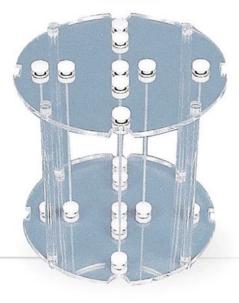







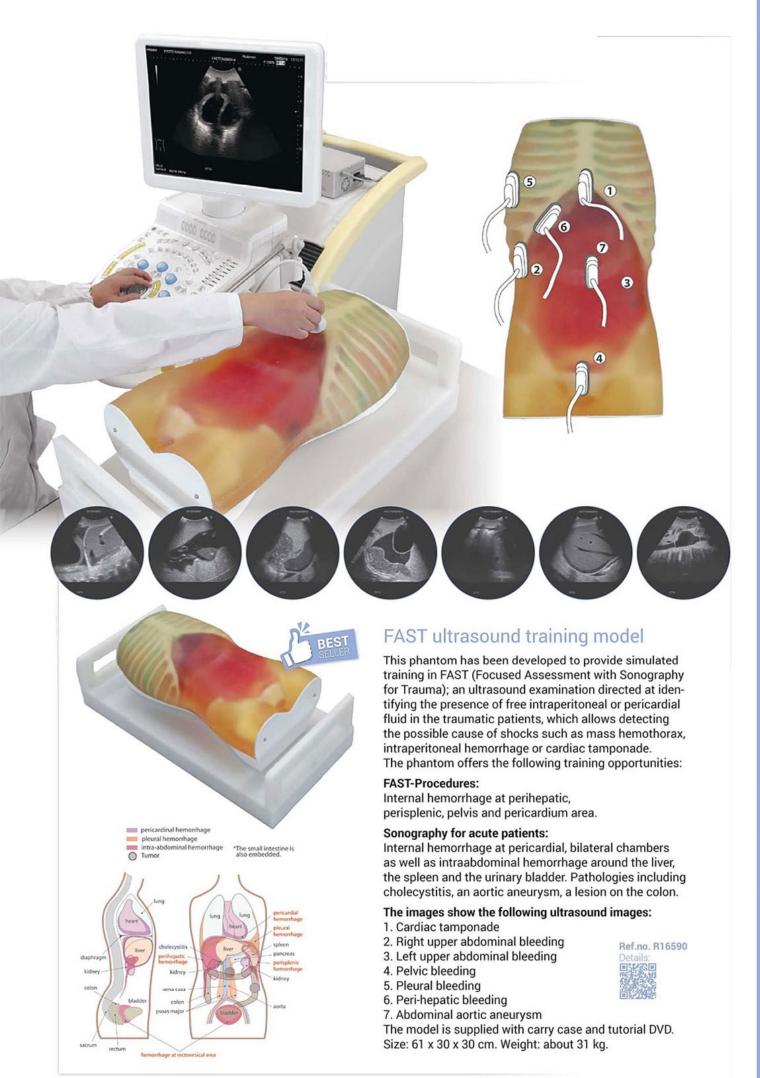

Pediatric FAST/ acute abdomen phantom

The world's first pediatric ultrasound torso phantom. Pediatric FAST/Acute Abdomen Phantom provides opportunities of hands on training in ultrasound that is a crucial modality for radiosensitive children. **Features:** The phantom includes life-size 2-year-old thoracoabdominal organs, a bone structure, free fluid to learn FAST procedures and pathologies that are commonly seen in pediatrics. With this phantom trainees can acquire skills in basics of pediatric abdominal ultrasound.


Pathologien:

- Internal hemorrhage at perihepatic, perisplenic, pelvis, and pericardium area
- Bowel intussusception, appendicitis and biliary dilatation

Specifications:

Set Includes: 1 ultrasound phantom, 1 storage case, 1 tutorial manual (DVD)




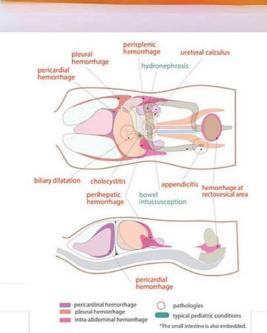







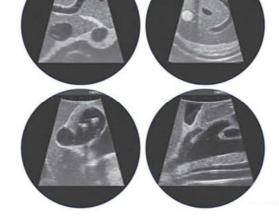

Abdominal intraoperative & laparoscopic ultrasound phantom

Features:

- √ Inanimate tool for training of a novice to demonstration by an expert.
- Detailed hepatobiliary, pancreatic and other abdominal anatomy meeting requirement for excellent training: open intraoperative scanning of the liver, biliary tract, pancreas; laparoscopic examination of the biliary system for screening of stones and evaluation of hepatic and pancreatic lesions, etc.
- Soft phantom materials allowing realistic probe manipulation.
- Various simulated lesions including biliary stones and cysts, solid tumors (hypoechoic, hyperechoic and target-appearance) in the liver, pancreas, spleen and kidneys.
- Detachable stomach and duodenum allowing various scanning methods of the bile duct and pancreas.
- Container allowing water-immersion scanning for both contact and stand-off techniques, simulating real abdominal intraoperative and laparoscopic scanning (no ultrasound gel required).
- Near real-size organs, structures and abnormal lesions.
- Container with phantom fits in the laparoscopic trainer box so that laparoscopic ultrasound of organs is possible under direct laparoscopic view.
- ✓ Durable long-life phantom materials.

The phantom includes:

the liver (segmental anatomy, portal and hepatic venous systems, ligamentum teres and ligamentum venosum), biliary tract (gallbladder, cystic duct, intrahepatic and extrahepatic bile ducts), pancreas (pancreatic duct), spleen, kidneys, detailed vascular structures (aorta, vena cava, celiac artery and its branches, portal vein and its branches, superior mesenteric vessels, renal vessels, etc.).

Pathology includes:

hepatic lesions (cystic and solid), gallbladder and bile duct stones, pancreatic tumors (one invading the portal vein), splenic lesions, both kidney lesions, and left adrenal tumor.





This high fidelity training model allows practice of ultrasound with existing ultrasound – machines. It includes the anatomy of the abdomen as well as many pathologies. The model includes the liver (Couinnaud's segments are visible), biliary tract, pancreas, spleen, kidneys, detailed vascular structures like aorta, vena cava, celiac artery and ist branches, portal vein and

its branches, superior mesenteric vessels, renal vessels etc. Multiple cysts and tumors in this model give various training opportunities for advanced examination. The phantom can be scanned from all sides like a real patient. Size: 28 x 25 x 18 cm, weight 12 kg.

posterior view
Rückansicht

Simulated cysts / künsdliche Zysten 10mm
Hypocholic / Roffevarm 10mm
Simulated Tumore 1, künstl. Tumore 10mm

Set ultrasound model and anatomical model

Ref.no. R16560-1 Details:

Details:





This model is perfect for the first education in ultrasound examination. It is a great tool for learning how to orientate in the upper abdomen and gives perfect images of all relevant organs and structures. In addition to liver, gallbladder, pancreas, spleen and vessels the lungs and ribs are represented in the model. The location of the organs and the thickness of the surrounding tissue are like in a real patient. Size: 28 x 25 x 18 cm, weight 12 kg.

Ultrasound model

Ultrasound model

Ref.no. R16560

Ref.no. R16570

etails:

Set ultrasound model and anatomical model

Ref.no. R16570-

Details:

Anatomical model for ultrasound education

This 20 part model of the upper abdominal organs represents exactly the anatomy that is inside the training models R16560 and R16570. This allows you to see the structures and organs three dimensional in front of you while you are scanning them in the training model. The single parts are: liver (can be separated into 8 segments), gall bladder, spleen, left kidney, vena cava, spine, large and small intestine, portal vein, bile duct and hepatic artery, pancreas, right kidney, abdominal aorta, hepatic vein and stomach. Size: 20 x 23 x 16 cm.







Female pelvic ultrasound phantom

Female pelvic ultrasound examination phantom facilitates teaching and learning how to practice both transvaginal and transabdominal ultrasound procedures using one's own device. Typical female pelvic pathologies are prepared for various training.

Training Skills:

- Female pelvic ultrasound screening both with transvaginal and transabdominal scanning
- Visualization of pathologies
- √ 3D ultrasound imaging restructuring

Features:

- Realistic pathology for transvaginal ultrasound training as well as transabdominal procedure.
- Excellent ultrasound image quality.
- Anatomically correct and life-like images.
- Universally compatible to any ultrasound machine.
- 2 types of interchangeable phantoms with various pathologies

Includes 2 phantoms:

Pathological Phantom

- Endometrial cancer, uterine fibroid
- dermoid cyst of ovary
- / bleeding at douglas cavum

Ectopic pregnancy phantom

Ectopic pregnancy in a fallopian tube







Ultrasound Neonatal Head Phantom with Hydrocephalus

This head phantom is designed to demonstrate abnormal anatomy, such as Hydrocephalus in which the shape of the skull is altered due to intercranial pressure. Training skills: Scanning of brain anatomy in Sagittal (Angled Parasagittal), Coronal und Transverse planes via any fontanel.

Anatomy:

- / Skull
- Anterior fontanel
- ✓ Posterior fontanel
- Hypertrophied lateral ventricle

Ref.no. R16810 Details:

Details:





More ultrasonic phantoms can be found on the pages:

Hip Sonography: Page 129
Breast Biopsy: Page 147
Testical Ultrasound: Page 148
Lumbar Puncture: from Page 186
CVC: from Page 201
Vascular Access: Page 205
Pericardicentesis/Thoracocentesis: Page 208
Ultrasound Bronchioscopy: Page 269

Intravesical Urine Volume Measurement Training Simulator

Pocket-sized ultrasound is becoming popular and start to be utilized in adult day care to check intravesical urine volume instead of catheterization which may cause urinary infection.

The following Skills can be trained:

- Handling and manipulation of transducers on bladder ultrasound.
- Scanning the bladder to measure urine volume and findings.

Includes 4 variations of interchangeable inserts feature different patient scenarios:

- / 50 ml urine
- / 150 ml urine
- / 300 ml urine
- √ Urinary retention/balloon catheter Size: 30 x 26 x 18 cm, Weight: approx. 3,2 kg

Ref.no. R16820 Details:





Rheumatoid Finger Phantom

Ultrasound examination is a key skill for early diagnosis on rheumatoid arthritis. For skills training in ultrasound examination on rheumatoid arthritis. Synovial thickening and increased synovial fluid are simulated on middle and ring finger. Size: 17 x 5 x 21cm, weight: 0.6 kg.

Fundamental Ultrasound Phantom

This oval phantom helps sonographers with basic trainings. It is a phantom for basic manipulation of the ultrasound transducer. The user is able to learn the recognition of shape, echogenicity, distance an to practice horizontal transducer works.

The user can learn to interpret 2D cross sectional images into a shape of 3D object. There are 10 objects embedded in the phantom. Learn how to adjust time-gain compensation to obtain suitable density, how to measure dimensions and distance. Objects with spheroidal and flat surfaces provide a variety of difficulty.



